

## #8500 ADULT ACUITY PROJECTOR SLIDE INSTRUCTIONS

The Stereo Optical Adult Acuity Test Slide has been designed for use with Marco, A.O. (Reichert), Woodlyn and similar projectors.


### CONVERT FOR USE IN A 10 FOOT LANE

Although initially configured for use in a 20 foot lane, the 8500 test slide can be easily converted for a 10 foot lane.

1. The glass slide is held in the frame by a spring in the bottom channel.
2. Remove the glass slide from the frame as follows:
  - ◆ Grasp slide with one hand and frame with the other.
  - ◆ Slide glass down within frame.
  - ◆ Tilt top of slide out and remove.
3. Rotate glass slide on a vertical axis.
4. Return slide to frame by reversing process step 2.

### CLEANING INSTRUCTIONS

Do not immerse slide in any liquid or spray liquid directly on slide, use a soft damp cloth to clean.

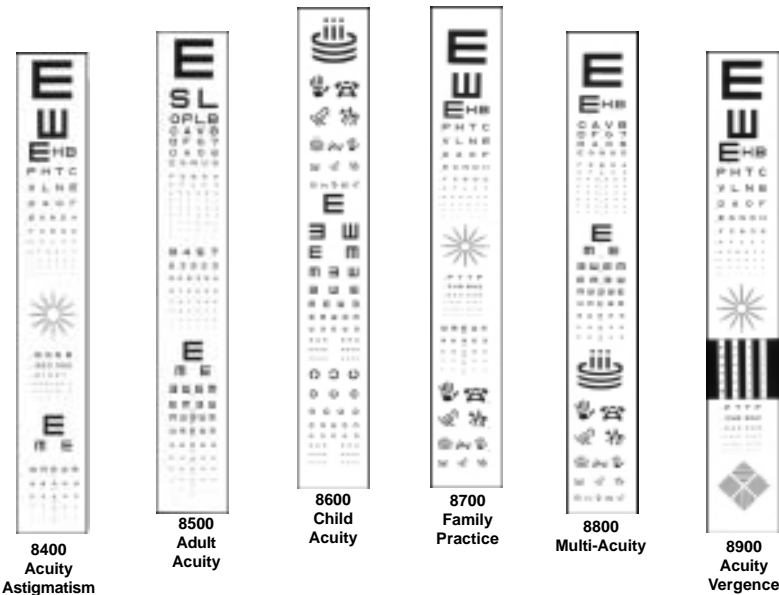

## #8500 ADULT ACUITY PROJECTOR SLIDE ANSWER KEY

### LETTER ACUITY

<u>LEVEL</u>	<u>ANSWER</u>
20/400	E
20/200	S L
20/100	O P L B
20/80	C A V 8
20/70	D F G 7
20/60	D A O 6
20/50	E G N U 5
20/40	F Z B D 4
20/30	O F L C 3
20/25	A P S O 2 5
20/20	E V O T Z 2
20/20	T E G A D M
20/20	H X F Z H G
20/20	T Z V E C L
20/15	O H P N T Z
20/10	E V O T S H

### NUMBER ACUITY

<u>LEVEL</u>	<u>ANSWER</u>
20/70	8 4 5 7
20/50	6 3 9 2 5
20/40	5 2 8 3 5 4
20/30	8 7 4 6 2 3
20/25	9 6 7 3 8 5
20/20	5 2 6 4 7 2
20/15	3 8 7 5 2 5
20/10	6 9 3 7 4 0

### TUMBLING "E" ACUITY

<u>LEVEL</u>	<u>ANSWER</u>
20/200	RIGHT
20/100	BOTTOM RIGHT
20/80	LEFT TOP RIGHT BOTTOM
20/70	RIGHT BOTTOM LEFT TOP
20/60	BOTTOM TOP LEFT TOP RIGHT
20/50	TOP BOTTOM RIGHT TOP BOTTOM
20/40	LEFT TOP BOTTOM RIGHT LEFT
20/30	RIGHT LEFT TOP BOTTOM RIGHT
20/25	BOTTOM TOP LEFT TOP LEFT
20/20	LEFT BOTTOM RIGHT BOTTOM BOTTOM
20/15	RIGHT TOP LEFT BOTTOM LEFT



**STEREO OPTICAL CO., INC.**

3539 N. Kenton Ave.  
Chicago, IL 60641  
800-344-9500  
Fax 773-777-4985

www.stereooptical.com  
sales@stereooptical.com