



# ACUITY- SUPPRESSION

#9 Rx VECTOGRAM®

STEREO OPTICAL CO.

Even though you see very well with either eye by itself, you may have some difficulty when using the eyes together. This target is designed to check the ability of each eye to read small print while both eyes are being used.

Line 4 is seen only by the left eye, and line 6 is seen only by the right eye. The other lines are seen by both eyes. Be sure neither line 4 nor 6 disappears, and normally they should be read with equal facility.

The small circle should appear to stand forward and the larger circle behind the letters. When using the reversible spectacles this position is reversed, as well as the eye relationship of lines 4 and 6.

This target may be placed in the stand with the Quoits to provide a reference point while moving the Quoits back and forth.

**NOTE:** Please store your Vectogram in a cool, dry place when not in use. High heat & humidity may cause fading.