



# COMPASS POINTS

#4 Rx VECTOGRAM\*

STEREO OPTICAL CO.

Through the compass window you see a hillside of Marysville Canyon in Southern Utah. Can you find the "Lady at the Organ" in the rock formation? With a pointer touch the tops of the many pinnacles. You will have to reach behind the Vectogram to do this. Look frequently at the + superimposed upon the picture, making sure that you see both the horizontal and the vertical lines.

Follow the compass marking around the window, stopping first at E to be sure you see the L and R in the long box. Now to S and study the Cowboys sitting on a rock in the Colorado River country of Northern Arizona. On to W and the New York skyline, looking back and forth from these skyscrapers to Nature's skyscrapers in Marysville Canyon. At N is a molecular pattern of balls. Repeat this circle several times, often glancing through to the distant hillside.

As in the other vectograms, vary the viewing distance as you study the picture.

**NOTE:** Please store your Vectogram in a cool, dry place when not in use. High heat & humidity may cause fading.