



# MOTHER GOOSE

#10 Rx VARIABLE VECTOGRAM<sup>®</sup>

STEREO OPTICAL CO.

Delightful characters from the tales of Mother Goose are going to help you develop the proper cooperation between your eyes.

Place the two vectographs in the two slots of the stand and start at the zero position on the scale. With the "3-D" spectacles on, if you do not see Humpty Dumpty's hat, Little Bo-Peep's staff, or Merry King Cole's bowl, you are not making use of your right eye. If you miss the cane, the sheep, or the pipe, it is your left eye which is not functioning properly. Conscious effort must be made to associate all these objects with their respective figures while the targets are separated to the positions on the scale determined by your eye specialist.

Humpty Dumpty should appear somewhat back of Little Bo-Peep, and Old King Cole a little forward. As you separate the targets on the numbered side of the scale, all the figures should come toward you and seem to get smaller. Take the pointer and touch each figure where it appears in space. When you separate them in the opposite direction to the letter side of the scale, the figures should go back and seem to get larger.

Separate the targets as far as possible, keeping the figures single and clear. Stay within any limits set by your doctor.

**NOTE:** Please store your Vectogram in a cool, dry place when not in use. High heat & humidity may cause fading.