



SPIRANGLE

#5 Rx VARIABLE VECTOGRAM®

STEREO OPTICAL CO.

This variable Vectogram contains numerous controls to insure constant awareness of binocular vision. Position the targets first at zero on the scale, and with your "3-D" glasses on observe the center square. You want to see three lines of small letters. If one eye is not functioning properly you may see only two. Spiraling forward from this square in a rectangular pattern is the jumbled alphabet connected by a pathway, which if interpreted properly, appears to come constantly out from the film toward you. At the end of this pathway is another square with three lines of small letters.

Adjacent to each square are circles each containing a large L and R, which stand for "Left" and "Right." If you do not see one of these, cover the opposite eye until you can see it, and then try to retain vision as you uncover the eye. If you have difficulty, separate the targets to a different position on the scale, or move closer or farther away.

All the letters of the alphabet are here so that you can play "Anagrams." Jump quickly from one letter to another. Each letter has a specific position in space relative to the small square around it. It may appear to come forward, be even with, or back. For example, in the word "boy," B comes forward, O is on plane, and Y is back. Be sure of the correct depth of each letter as you spell out different words.

Follow your eye specialist's instructions concerning positioning of targets and methods of training.

NOTE: Please store your Vectogram in a cool, dry place when not in use. High heat & humidity may cause fading.