



# STEREO TEST

#11 Rx VARIABLE VECTOGRAM<sup>®</sup>

STEREO OPTICAL CO.

When the eyes are working together properly, the word "fusion" describes the merging of the two images from the two eyes into one image in the brain. But the mind may sense the image from only one eye, suppressing the other one, and seem to be seeing properly. This Vectogram detects quickly if the two eyes and the brain are correctly fusing the images as some of the numbered circles will stand forward stereoscopically in three dimensions, while they will appear on the same plane as the others if only one eye is used. Each successive line is a graded step. It becomes increasingly more difficult to select the circle which stands forward from the rest.

Line	Correct Circle	Approximate Stereo Percentage at 16"
A	4	10%
B	2	20%
C	5	40%
D	3	60%
E	4	75%
F	2	90%

Below the circles is a square with an "L" that can be seen only by the left eye and a circle with an "R" that can be seen only by the right eye. They should not disappear as you train within the limits your doctor recommends. If one does disappear, cover the other eye momentarily until you again see it and then consciously try to retain them both.

**NOTE:** Please store your Vectogram in a cool, dry place when not in use. High heat & humidity may cause fading.