

#8400 ACUITY PROJECTOR SLIDE INSTRUCTIONS

#8400 ACUITY PROJECTOR SLIDE ANSWER KEY

The Stereo Optical Acuity Test Slide has been designed for use with Marco, A.O. (Reichert), Woodlyn and similar projectors.


CONVERT FOR USE IN A 10 FOOT LANE

Although initially configured for use in a 20 foot lane, the 8400 test slide can be easily converted for a 10 foot lane.

1. The glass slide is held in the frame by a spring in the bottom channel.
2. Remove the glass slide from the frame as follows:
 - ♦ Grasp slide with one hand and frame with the other.
 - ♦ Slide glass down within frame.
 - ♦ Tilt top of slide out and remove.
3. Rotate glass slide on a vertical axis.
4. Return slide to frame by reversing process step 2.

CLEANING INSTRUCTIONS

Do not immerse slide in any liquid or spray liquid directly on slide, use a soft damp cloth to clean.

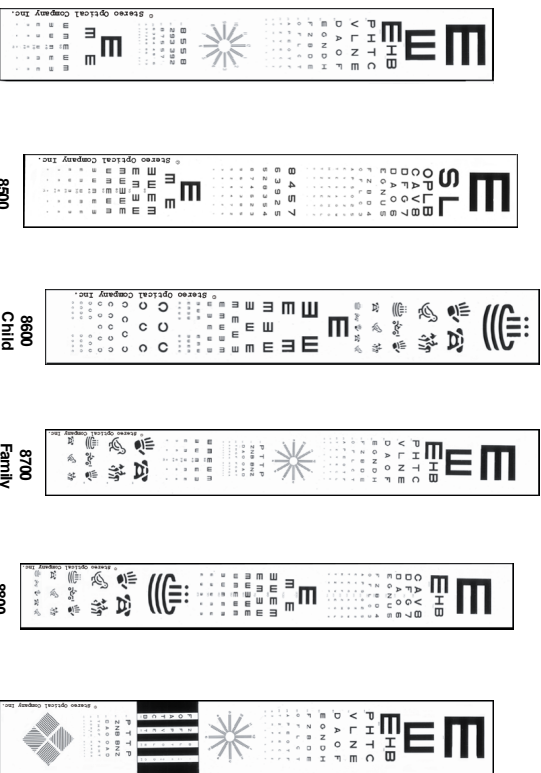

LETTER ACUITY

LEVEL	ANSWER
20/400	RIGHT
20/300	UP
20/200	E
20/100	H B
20/80	P H T C
20/70	V L N E
20/60	D A O F
20/50	E G N D H
20/40	F Z B D E
20/30	O F L C T
20/25	A P E O T F
20/20	T Z V E C L
20/15	O H P N T Z
20/10	E V O T S H F

MIRROR KEY NUMBER ACUITY

LEVEL	ANSWER
20/50	8 5 - 5 8
20/40	2 9 3 - 3 9 2
20/30	8 7 5 - 5 7 8
20/20	6 3 9 4 - 4 9 3 6
20/15	3 7 2 6 8 - 8 6 2 7 3
20/10	3 2 8 5 3 7 - 7 3 5 8 2 3

TUMBLING "E" ACUITY

LEVEL	ANSWER
20/200	RIGHT
20/100	BOTTOM
20/50	TOP BOTTOM RIGHT TOP BOTTOM
20/40	LEFT TOP BOTTOM RIGHT LEFT
20/30	RIGHT LEFT TOP BOTTOM RIGHT
20/20	LEFT BOTTOM RIGHT BOTTOM BOTTOM
20/15	RIGHT TOP LEFT BOTTOM LEFT



STEREO OPTICAL CO., INC.
8600W, Catalpa Avenue, Suite 703
Chicago, IL 60656

800-344-9500
Fax 773-867-0388
www.stereooptical.com
sales@stereooptical.com

© 2017 Stereo Optical Co., Inc. All rights reserved. Unless indicated otherwise, all trademarks are the property of Stereo Optical Co., Inc. in the United States and other countries.