

#8600 CHILD ACUITY PROJECTOR SLIDE INSTRUCTIONS

The Stereo Optical Acuity Test Slide has been designed for use with Marco, A.O. (Reichert), Woodlyn and similar projectors.

CONVERT FOR USE IN A 10 FOOT LANE

Although initially configured for use in a 20 foot lane, the 8600 test slide can be easily converted for a 10 foot lane.

1. The glass slide is held in the frame by a spring in the bottom channel.
2. Remove the glass slide from the frame as follows:
 - ◆ Grasp slide with one hand and frame with the other.
 - ◆ Slide glass down within frame.
 - ◆ Tilt top of slide out and remove.
3. Rotate glass slide on a vertical axis.
4. Return slide to frame by reversing process step 2.

CLEANING INSTRUCTIONS

Do not immerse slide in any liquid or spray liquid directly on slide, use a soft damp cloth to clean.

#8600 CHILD ACUITY PROJECTOR SLIDE ANSWER KEY

ALLEN PRESCHOOL ACUITY


<u>LEVEL</u>	<u>ANSWER</u>
20/200	CAKE
20/100	HAND PHONE
20/80	BIRD HORSE
20/60	CAKE CAR HAND
20/40	PHONE BIRD HORSE
20/30	CAKE CAR HAND PHONE BIRD

TUMBLING "E" ACUITY

<u>LEVEL</u>	<u>ANSWER</u>
20/200	RIGHT
20/160	LEFT TOP
20/125	RIGHT BOTTOM
20/100	BOTTOM LEFT TOP
20/80	LEFT TOP RIGHT
20/60	BOTTOM RIGHT TOP LEFT
20/50	RIGHT BOTTOM TOP LEFT BOTTOM
20/40	TOP LEFT RIGHT TOP LEFT
20/30	LEFT TOP BOTTOM - RIGHT BOTTOM TOP
20/25	RIGHT BOTTOM LEFT TOP - LEFT TOP RIGHT BOTTOM
20/20	BOTTOM RIGHT TOP LEFT - TOP LEFT BOTTOM RIGHT

LANDOLT "C" ACUITY

<u>LEVEL</u>	<u>ANSWER</u>
20/100	BOTTOM LEFT TOP
20/80	LEFT TOP RIGHT
20/60	BOTTOM RIGHT TOP LEFT
20/50	RIGHT BOTTOM TOP LEFT BOTTOM
20/40	TOP LEFT RIGHT TOP LEFT
20/30	LEFT TOP BOTTOM - RIGHT BOTTOM TOP
20/25	RIGHT BOTTOM LEFT TOP - LEFT TOP RIGHT BOTTOM
20/20	BOTTOM RIGHT TOP LEFT - TOP LEFT BOTTOM RIGHT

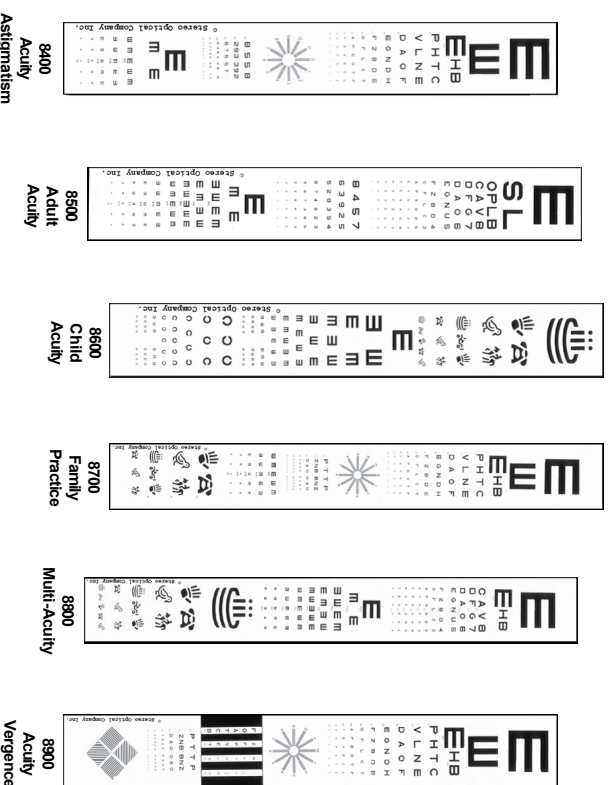

STEREO OPTICAL CO., INC.
8600W, Catalina Avenue, Suite 703
Chicago, IL 60656
800-344-9500

www.stereooptical.com
sales@stereooptical.com
Fax 773-867-0388

© 2017 Stereo Optical Co., Inc. All rights reserved. Unless indicated otherwise, all trademarks are the property of Stereo Optical Co., Inc. in the United States and other countries.