

#8700 FAMILY PRACTICE PROJECTOR SLIDE INSTRUCTIONS

The Stereo Optical Acuity Test Slide has been designed for use with Marco, A.O. (Reichert), Woodlyn and similar projectors.

CONVERT FOR USE IN A 10 FOOT LANE

Although initially configured for use in a 20 foot lane, the 8700 test slide can be easily converted for a 10 foot lane.


1. The glass slide is held in the frame by a spring in the bottom channel.
2. Remove the glass slide from the frame as follows:
 - ♦ Grasp slide with one hand and frame with the other.
 - ♦ Slide glass down within frame.
 - ♦ Tilt top of slide out and remove.
3. Rotate glass slide on a vertical axis.
4. Return slide to frame by reversing process in step 2.

CLEANING INSTRUCTIONS

Do not immerse slide in any liquid or spray liquid directly on slide, use a soft damp cloth to clean.



8400
Acuity
Astigmatism

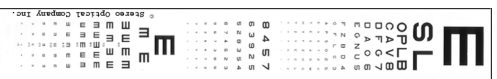

8800
Adult
Acuity



8600
Child
Acuity



8700
Family
Practice



8800
Multi-Acuity



8900
Acuity
Vergence

#8700 FAMILY PRACTICE PROJECTOR SLIDE ANSWER KEY

LETTER ACUITY

<u>LEVEL</u>	<u>ANSWER</u>
20/400	RIGHT
20/300	UP
20/200	E
20/100	H B
20/80	P H T C
20/70	V L N E
20/60	D A O F
20/50	E G N D H
20/40	F Z B D E
20/30	O F L C T
20/25	A P E O T F
20/20	T Z V E C L
20/15	O H P N T Z
20/10	E V O T S H F

ASTIGMATIC CLOCK

MIRROR KEY NUMBER ACUITY

<u>LEVEL</u>	<u>ANSWER</u>
20/50	P T - T P
20/40	Z N B - B N Z
20/30	D A O - O A D
20/20	T H G F - F G H T
20/15	A D M X H - H X M D A
20/10	K H N Z D B - B D Z N H K

TUMBLING "E" ACUITY

<u>LEVEL</u>	<u>ANSWER</u>
20/50	TOP BOTTOM RIGHT TOP BOTTOM
20/40	LEFT TOP BOTTOM RIGHT LEFT
20/30	RIGHT LEFT TOP BOTTOM RIGHT
20/20	LEFT BOTTOM RIGHT BOTTOM BOTTOM
20/15	RIGHT TOP LEFT BOTTOM LEFT

ALLEN PRESCHOOL ACUITY

<u>LEVEL</u>	<u>ANSWER</u>
20/100	HAND PHONE
20/80	BIRD HORSE
20/60	CAKE CAR HAND
20/40	PHONE BIRD HORSE



STEREO OPTICAL CO., INC.

8600W. Catalpa Avenue, Suite 703
Chicago, IL 60656

800-344-9500 www.stereooptical.com
Fax 773-867-0388 sales@stereooptical.com