

#8800 MULTI-ACUITY PROJECTOR SLIDE INSTRUCTIONS

#8800 MULTI-ACUITY PROJECTOR SLIDE ANSWER KEY

The Stereo Optical Multi-Acuity Test Slide has been designed for use with Marco, A.O. (Reichert), Woodlyn and similar projectors.


CONVERT FOR USE IN A 10 FOOT LANE

Although initially configured for use in a 20 foot lane, the 8800 test slide can be easily converted for a 10 foot lane.

- The glass slide is held in the frame by a spring in the bottom channel.
- Remove the glass slide from the frame as follows:
 - ♦ Grasp slide with one hand and frame with the other.
 - ♦ Slide glass down within frame.
 - ♦ Tilt top of slide out and remove.
- Rotate glass slide on a vertical axis.
- Return slide to frame by reversing process in step 2.

CLEANING INSTRUCTIONS

Do not immerse slide in any liquid or spray liquid directly on slide, use a soft damp cloth to clean.

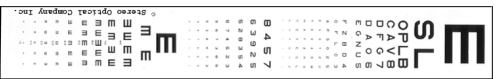

LETTER ACUITY

LEVEL	ANSWER
20/400	E
20/200	H B
20/100	C A V 8
20/80	D F G 7
20/70	D A O 6
20/60	E G N U 5
20/50	F Z B D 4
20/40	O F L C 3
20/30	A P S O 2 5
20/25	E V O T Z 2
20/20	T E G A D M
20/20	H X F Z H G
20/20	T Z V E C L
20/15	O H P N T Z

TUMBLING "E" ACUITY

LEVEL	ANSWER
20/200	RIGHT
20/100	BOTTOM RIGHT
20/100	LEFT TOP RIGHT BOTTOM
20/80	RIGHT BOTTOM LEFT TOP
20/70	BOTTOM TOP LEFT TOP RIGHT
20/60	TOP BOTTOM RIGHT TOP BOTTOM
20/50	LEFT TOP BOTTOM RIGHT LEFT
20/40	RIGHT LEFT TOP BOTTOM RIGHT
20/30	BOTTOM TOP LEFT TOP LEFT
20/25	BOTTOM TOP LEFT TOP LEFT

ALLEN PRESCHOOL ACUITY

LEVEL	ANSWER
20/200	CAKE
20/200	HAND PHONE
20/100	BIRD HORSE
20/80	CAKE CAR HAND
20/60	PHONE BIRD HORSE
20/40	CAKE CAR HAND PHONE BIRD
20/30	CAKE CAR HAND PHONE BIRD



STEREO OPTICAL
86 W. RUBENW/FOV/FAV/JF
Chicago, IL 606

www.stereooptical.com
sales@stereooptical.com

4URSPFOQDBN8 ODMNSHUTSTFSVFBOMFTTODRUFPUFS3F BMMUSBFNBSTTRBF
PGBLZ PGUFSFPQDBN8 CDDUFRQUJE 4UBURFOEPUFSDPVOUSBT