

# #8900 ACUITY VERGENCE PROJECTOR SLIDE INSTRUCTIONS

The Stereo Optical Acuity Test Slide has been designed for use with Marco, A.O. (Reichert), Woodlyn and similar projectors.

## CONVERT FOR USE IN A 10 FOOT LANE

Although initially configured for use in a 20 foot lane, the 8900 test slide can be easily converted for a 10 foot lane.


1. The glass slide is held in the frame by a spring in the bottom channel.
2. Remove the glass slide from the frame as follows:
  - ◆ Grasp slide with one hand and frame with the other.
  - ◆ Slide glass down within frame.
  - ◆ Tilt top of slide out and remove.
3. Rotate glass slide on a vertical axis.
4. Return slide to frame by reversing process step 2.

## CLEANING INSTRUCTIONS

Do not immerse slide in any liquid or spray liquid directly on slide, use a soft damp cloth to clean.



8400  
Acuity  
Astigmatism

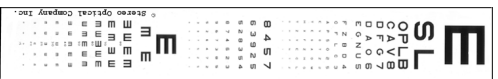

8800  
Adult  
Acuity



8600  
Child  
Acuity



8700  
Family  
Practice



8800  
Multi-Acuity



8900  
Acuity  
Vergence

# #8900 ACUITY PROJECTOR SLIDE ANSWER KEY

## LETTER ACUITY

LEVEL	ANSWER
20/400	RIGHT
20/300	Up
20/200	E
20/100	H B
20/80	P H T C
20/70	V L N E
20/60	D A O F
20/50	E G N D H
20/40	F Z B D E
20/30	O F L C T
20/25	A P E O T F
20/20	T Z V E C L
20/15	O H P N T Z
20/10	E V O T S H F

## VERTICAL LINES FOR VERGENCE (From Top to Bottom)

LEVEL	ANSWER
20/40	F O A T C B
20/30	Z F P V E T
20/20	B L T O L S

## TUMBLING "C" ACUITY (Which direction does C open to?)

LEVEL	ANSWER
20/10	EAST
20/10	SOUTHEAST
20/15	SOUTH
20/15	SOUTHWEST
20/20	WEST
20/20	NORTHWEST

## MIRROR KEY NUMBER ACUITY

LEVEL	ANSWER
20/50	8 5 - 5 8
20/40	2 9 3 - 3 9 2
20/30	8 7 5 - 5 7 8
20/20	6 3 9 4 - 4 9 3 6
20/15	3 7 2 6 8 - 8 6 2 7 3
20/10	3 2 8 5 3 7 - 7 3 5 8 2 3

## ASTIGMATIC SQUARE (Which way do lines run?)

TOP SQUARE:	HORIZONTAL
BOTTOM SQUARE:	VERTICAL
RIGHT SQUARE:	NORTHWEST TO SOUTHEAST
LEFT SQUARE:	SOUTHWEST TO NORTHEAST



**STEREO OPTICAL CO., INC.**  
8623 W. Bryn Mawr Ave., Suite 502  
Chicago, IL 60631  
800-344-9500  
Fax: 773-867-0388

www.stereooptical.com  
sales@stereooptical.com