



## STEREO OPTICAL

To Whom It May Concern:

Thank you very much for your recent inquiry on cleaning and care instructions for our Stereotests. The Stereotests should be placed in a cool, dry place when not in use. High heat and humidity will cause the stereotest to fade. Stereotests should be cleaned in the following manner:

Liquid cleaning solution or alcohol should never be sprayed directly on the test films or on the polarized lenses. (i.e. picture of the Fly, Butterfly, shapes or glasses used for the patient to look through)

If you use liquid cleaning solution, spray it on to a lint-free cloth. **Do not** over saturate the cloth with the solution as any left over liquid may cause the Stereotest and polarized lens film to shrink and/or bubble.

Note: We highly encourage the use of alcohol to clean the test film and polarized lenses, as this is a good way to ensure that any bacteria is not left on the book or glasses.

If you have any further questions or concerns about proper care and maintenance of these products please do not hesitate to contact us again.

Best regards,

**Stereo Optical Company, Inc.**  
8600 W Catalpa Ave suite 703  
Chicago, IL 60656  
Tollfree: 800.344.9500  
**Phone: 773.867.0380**  
Fax: 773.867.0388

### STEREO OPTICAL COMPANY, INC.

8600 W. Catalpa Ave., Suite 703 Chicago, IL 60656 U.S.A. Phone: 1-773-867-0380 1-800-344-9500 Fax: 1-773-867-0388  
E-mail: [sales@stereooptical.com](mailto:sales@stereooptical.com) Website: [www.stereooptical.com](http://www.stereooptical.com)